




This sponge

By eliminating the disadvantages of earlier methods, the Triosorb Sponge has achieved a real breakthrough in thyroid testing. **It is an in vitro test unmatched in accuracy, speed and convenience.**

Accuracy: Because factors such as red blood cells and exogenous iodine have been eliminated from consideration in the Triosorb Test, it is unmatched in accuracy.

Speed: With only 3 washes and no need for double pipettings, shakers, or incubators, the Triosorb Test can be

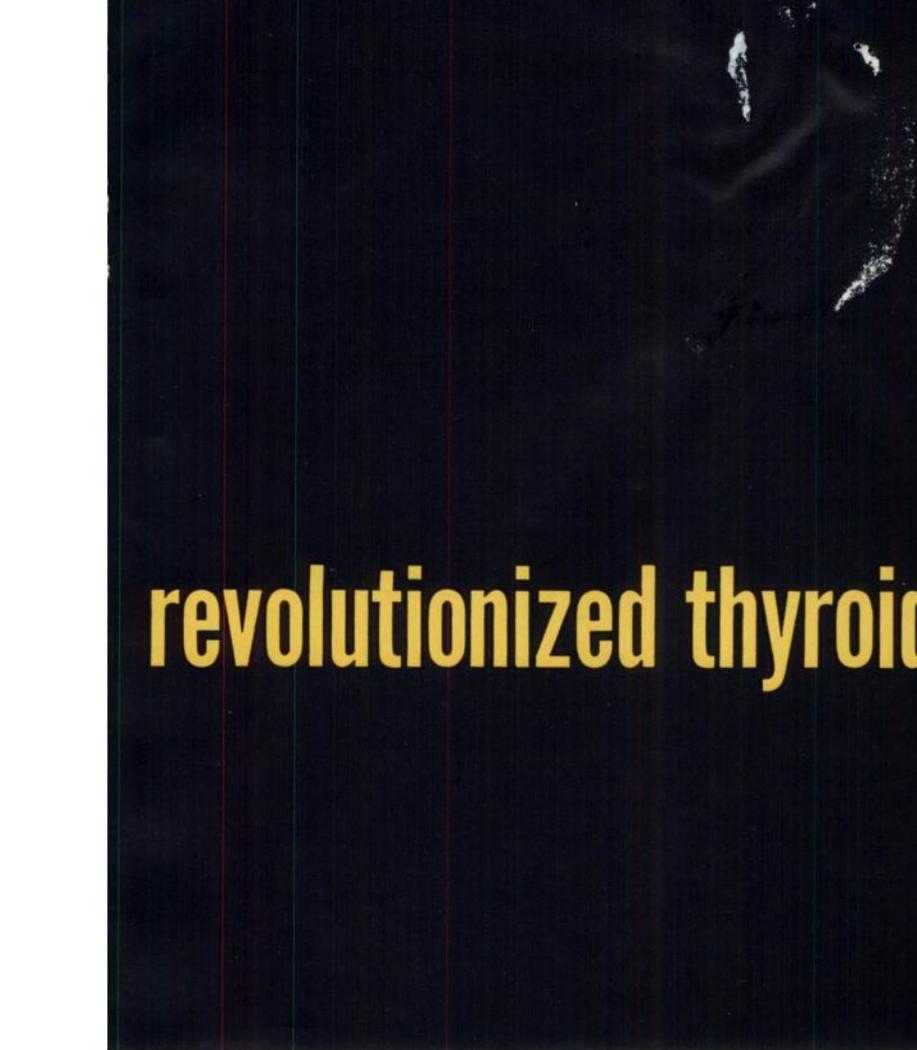

revolutionized thyroid testing!

more rapidly performed than any other T-3 test.

Convenience: Triosorb is in a disposable kit ready for immediate use at room temperature, making it the simplest and most convenient thyroid function test to perform.

McAdams* reported that "The resin sponge (Triosorb) technique is superior to the erythrocyte method for performing the I^{131} T3 test in terms of simplicity, convenience and elimination of errors characteristic of the erythrocyte procedure."

Triosorb is available to all doctors, hospitals and clinical laboratories—AEC licensing is not required. Because Triosorb will enable far more screenings to be performed, this procedure may soon become as standard as today's blood counts and urinalyses.




*McAdams, G. B. and Reinfrank, R. F., *Jrnl. Nuclear Med.*, 5:112, Feb., 1964.

TRIOSORB[®]
T-3 DIAGNOSTIC KIT
ABBOTT LABORATORIES NORTH CHICAGO, ILL.

FOR BIOLOGICAL RESEARCH

... CLINICAL DIAGNOSIS

... SPECIAL STUDIES

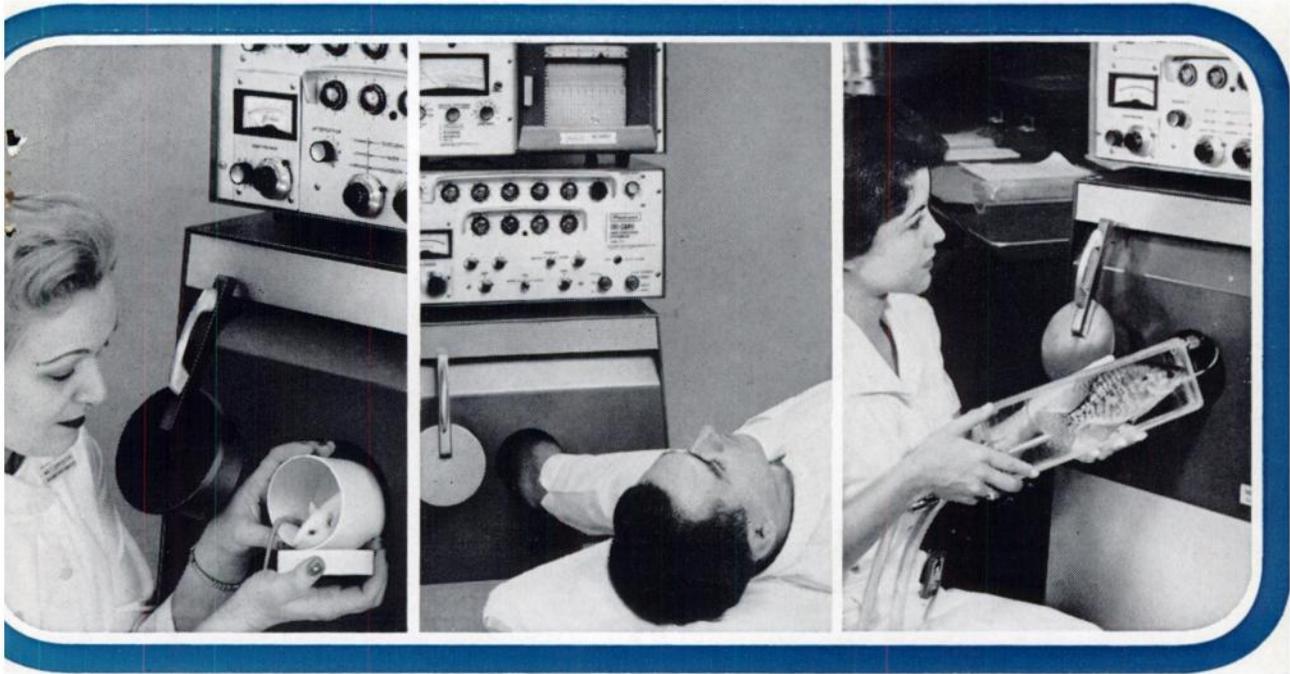

ARMAC® provides rapid, accurate measurement of radioactivity

The Armac Scintillation Detector is ideally suited for *in vivo* research using gamma-emitting tracers. Its 4-pi counting geometry and high counting efficiency require the administration of only small doses of radioactivity, yet assure significant measurements in short counting periods.

- In biological research Armac can be used to count radioactivity in small laboratory animals in the study of retention and excretion of various gamma-emitting isotopes. Because only small doses of radioactivity are required, sacrificing is unnecessary and the animal acts as its own control for repeated experimentation.
- In clinical research Armac is useful in nutritional

studies and other applications where changes in the amount of circulating radioactivity in the blood can be directly related to other body functions. These changes can be measured accurately and rapidly using the blood-rich portion of the patient's forearm positioned in the Armac's counting chamber.


- In special studies Armac can be used to quickly and accurately determine the presence of low levels of radioactivity in bulk samples of meat, milk, water, blood, wastes, soil, or tissue with little or no sample preparation. The photograph at right, above, shows an Armac Detector counting radioactivity in a live fish in flowing stream water. (Courtesy Oak Ridge National Laboratory.)

A complete counting system includes the Model 446 Armac Scintillation Detector and any of several Packard Spectrometers, control units and ratemeters. There are more than ten Armac Detector Systems available to meet your specific counting requirements. Choose visual or recorded data output; digital or analog data presentation. Any system can be readily modified or expanded to meet changing research requirements. Special systems incorporating automatic sample changers are also available.

For detailed information contact your Packard Sales Engineer, or write for Bulletin 1024.

Packard

PACKARD INSTRUMENT COMPANY, INC.
2200 WARRENVILLE ROAD • DOWNERS GROVE, ILLINOIS 60515
TELEPHONE: 312/969-6000

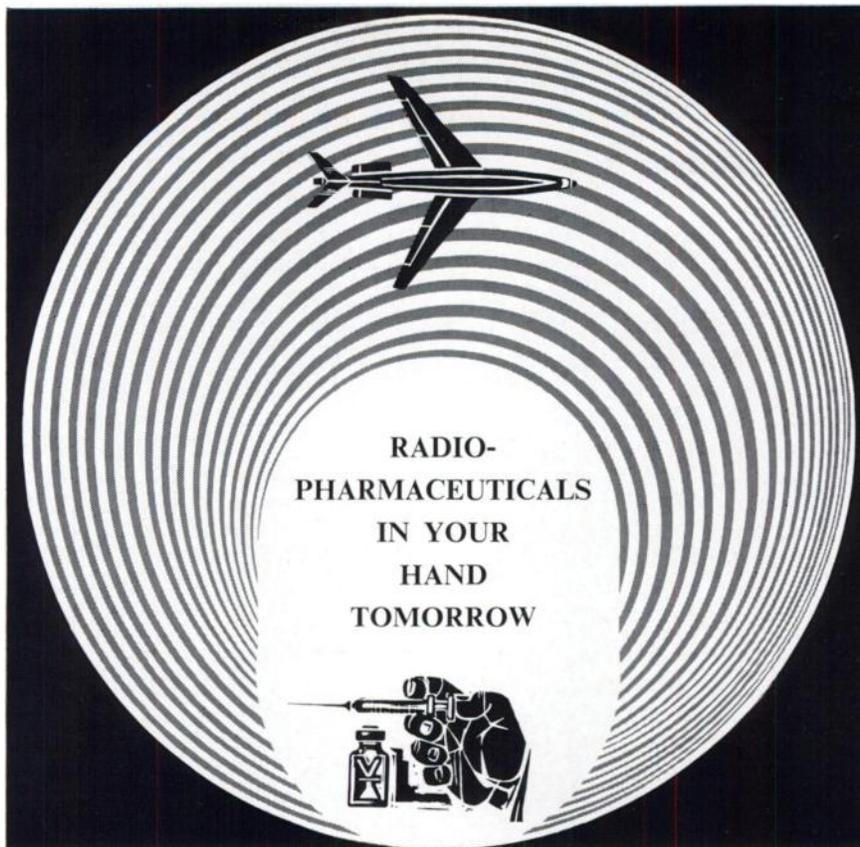

The Volk Radiochemical Company has been producing radiopharmaceuticals and radiochemicals for human and research use for the past ten years. This is our only business. Continued technological improvements and innovations have been constant goals as exemplified by our pioneering production of Iodine-125 as a useful medical isotope and our introduction of the "Silver Saddle" which removes free iodide from iodinated organic compounds such as Hippuran.

Twenty separate individual isotopic dose products are available in standard and special potencies.

CONVENIENT—to the world's busiest airport (Chicago's O'Hare) insures delivery tomorrow via Air Express to any one of 21,000 U. S. cities.

<p>Collect call ordering to our Skokie, Illinois laboratory (area code 312 673-3760)</p>		<p>or Burbank, California laboratory (area code 213 849-6023) assures you that your orders are handled as specified.</p>
--	--	--

Write for the 1965 Volk Radiomedicines catalog.

Volk RADIOCHEMICAL COMPANY



CHICAGO—8260 Elmwood Ave., Skokie, Ill.
LOS ANGELES—803 N. Lake St., Burbank, Calif.
WASHINGTON—P.O. Box 335, Silver Spring, Md.
NEW YORK—P.O. Box 345, New York, N. Y.

Duphar Cyclotron on its way to Petten, Holland

Philips-Duphar will soon be able to operate its own cyclotron in the new Isotope Laboratory at the Reactor Centre, Petten, Holland. This cyclotron will be the first one in the world to be operated by a private firm and will be used for the production of carrier-free radioactive isotopes exclusively.

The cyclotron is of the isochronous type, manufactured in the Philips Works at Eindhoven. The truck in the picture carried its 90-ton cloverleaf magnet during a 2-night transport from Eindhoven to Petten.

Representatives in 70 countries. 480 minute service*. Daily shipments to every part of the world.

Delivery program besides cyclotron isotopes: reactor isotopes, C14/H3 compounds, industrial sources and Nuclear Pharmaceuticals. Catalogues and pricelists on request.

* Orders for stock-items received before 10 a.m. can be on the way at 6 p.m.



**N.V. PHILIPS-DUPHAR
AMSTERDAM HOLLAND**

151 Apollolaan


Telegrams: Vitamine-Amsterdam

carrier-free cyclotron isotopes:

Be7 - Na22 - Ti44/Sc44 - V48 - Fe55 - Co57 -
Ge68/Ga68 - Se72/As72 - As 74 - Sr82/Rb82 -
Sr85 - Y87/Sr87m - Nb95/Zr95 - Cd109 -
Sn113/In113 - Te118/Sb118 - I124 - W178/
Ta178 - Bi206



TECHNETIUM-99m ISOTOPE GENERATOR

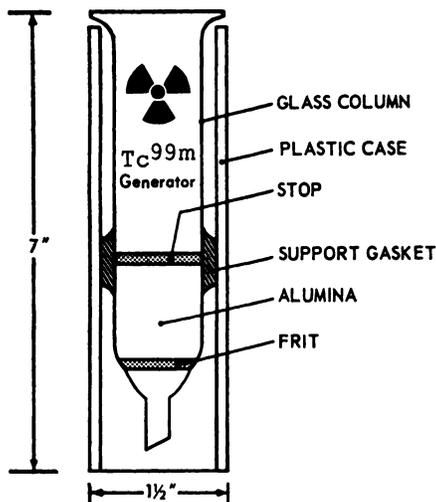

- for rapid, low-dose scanning
- over 100 mc Tc^{99m} per week (milked daily)
- individually Mo⁹⁹ leak tested before shipment
- delivered every Monday AM

Application:

Technetium-99m is an excellent radionuclide for scanning. Its only significant emission is a readily collimated 140 kev gamma photon and its short, 6 hour half life minimizes patient exposure.

Rationale:

Technetium-99m is continually produced in NENC's isotope generator by the decay of its radioactive parent, molybdenum-99 (Mo⁹⁹ → Tc^{99m}). The molybdenum (t_{1/2} = 66 hours) is adsorbed on an alumina column. Technetium-99m is removed as the pertechnetate ion (TcO₄⁻) by eluting with ~ 20 ml of physiological saline solution.

Output:

The amount of technetium-99m which can be eluted at one time is determined by the total amount of molybdenum-99 on the column, and by the time elapsed since the previous elution. A column containing 100 mc of Mo⁹⁹ will yield about 50 mc of Tc^{99m} 24 hours after the previous elution.

Mo-99 Leakage:

Typically less than 0.1 μc Mo⁹⁹ per wash. Each column is tested prior to shipment and does not leak more than 1 part in 10⁴.

Delivery:

Shipped Saturdays via air express for Monday morning delivery. Precalibrated for 8:00 A.M. Monday E.S.T.. Other arrangements on request.

Shielding:

Columns are shipped in a disposable lead container. A 3.5" diameter lead shield (1" walls) with bottom exit port is available for in-laboratory shielding. Price: \$50 (Delivered)

License Requirements:

Orders for NENC generators must be accompanied by a copy of the purchaser's "AEC-313a License for Human Use of Byproduct Material" or the standard AEC or state license plus a statement certifying that the radioisotope will not be administered to humans.

Price Schedule:


IGT-100 (100mc Mo ⁹⁹) \$ 95
IGT-200 (200mc Mo ⁹⁹) \$150

Weekly, biweekly, or monthly shipments.

F.O.B. Boston, Gross shipping weight 12 lbs.



NEW ENGLAND NUCLEAR CORP.
575 ALBANY STREET, BOSTON, MASS. 02118 AREA CODE 617 | 428-7311

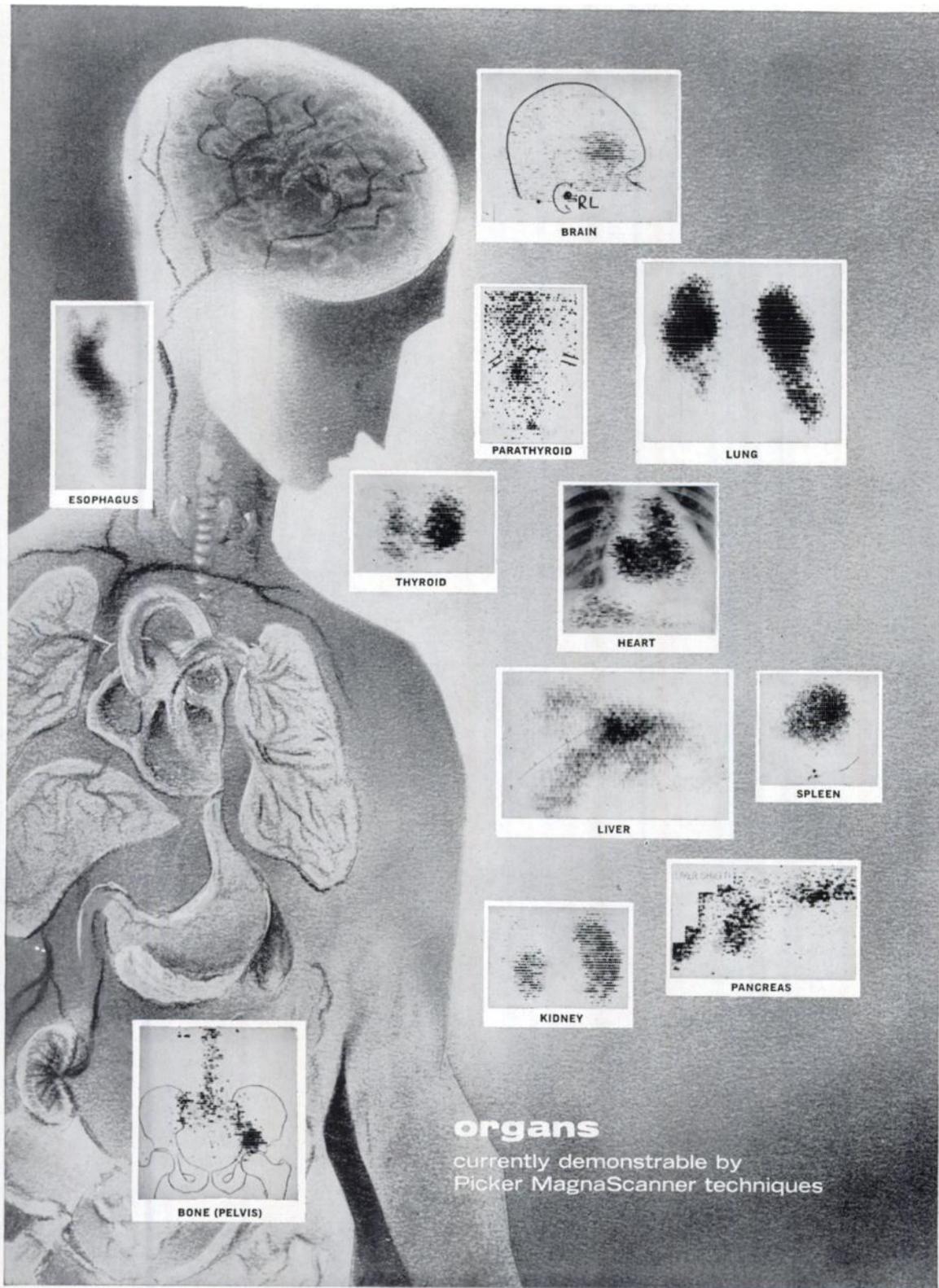



 **MEDOTOPES[®]**
SQUIBB RADIOPHARMACEUTICALS

Medotopes reflect the latest developments in nuclear medicine. All provide the utmost in safety and convenience. All have unique packaging safeguards so that direct contact is never required. Exclusive lead shield enclosures are fitted with bottle caps that unscrew automatically. Saf-Tag[®] vials and bottles are carefully encased and double protected by transparent, shatterproof plastic coatings, and shipping cartons have convenient "pull-tab" openers. And, each preparation is custom-handled, each delivery custom-routed by Squibb Traffic Service. Access to three major airports expedites shipment.

Squibb Radiopharmaceuticals are available to the AEC-licensed physician. For full information, write to Professional Service Dept., Squibb, 745 Fifth Avenue, New York 22, N. Y.

SQUIBB
Squibb Quality—the Priceless Ingredient
SQUIBB DIVISION **Olin**



Large Crystal Radioisotope Scanners


OHIO-NUCLEAR
Model 54



FEATURES:

- Soundly engineered—research proven in use.
- 200 inches (500 cm) per minute scanning speeds.
- Choice of above or beneath table scanning.
- Dual Scanner capability—now or later.
- Gammagraphic Photoscan & Dot-tapper.
- Prompt service by capable factory engineers.
- 5-inch crystal scanner price—\$16,750.
- 8-inch crystal scanner price—\$20,700.

For Full Information WRITE OR CALL US • WE BELIEVE YOU WILL BE GLAD YOU DID.

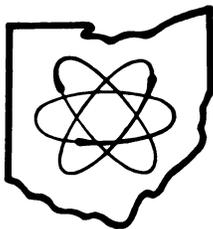

OHIO-NUCLEAR, INC.

1725 FALL AVENUE

CLEVELAND, OHIO

216 - 621-8477

**looking over
nuclear
instruments?**



**one of these top researchers
belongs on your staff.**

RESUME:

**GAMMA/GUARD CONSOLE WELL
COUNTING SYSTEM**

OCCUPATION:

Automatic gamma scintillation test tube counting.

SPECIAL TALENTS:

Shares all the exclusive capabilities of Omni/Guard systems — wherever applicable. In addition, Gamma/Guard offers a variety of sample capacities and detector sizes.

FAMILY:

From the oldest, from the first family in nuclear instrumentation —

RESUME:

**4 π AND 2 π CHROMATOGRAM
SCANNERS**

OCCUPATION:

Complete scanning systems for paper or thin layer chromatography.

SPECIAL TALENTS:

Can integrate areas under curve (recycling integrate) • ratemeter output connector for system expansion • improved background and tracking ability.

VERSATILITY:

Buy the system for either scanning mode, and convert in minutes to alternate mode through attachments purchased as required.

FAMILY:

From the oldest, from the first family in nuclear instrumentation —

RESUME:

OMNI/GUARD SCALER

OCCUPATION:

Works as part of ultra-low background counting system in conjunction with detector and sample changer.

SPECIAL TALENTS:

Anti-coincidence circuits, decade scaler, electronic timer, stable high voltage, data printout — lister or computer, transistor circuits throughout.

FAMILY:

From the oldest, from the first family in nuclear instrumentation —

For more information on the Nuclear Instrument story, phone your local sales office, or write direct to Waltham.



TRACERLAB

A Division of Laboratory For Electronics, Inc.
WALTHAM, MASSACHUSETTS 02154

Film Badge Service • Health Physics • Bioassays • Sources • Nuclear Instrumentation • Radiochemicals
Radioactive Waste Disposal • Radiation Monitoring Instrumentation • Isotope Applications


A Valuable Addition To Your Professional Library

Journal of

NUCLEAR



*Official Publication
Society of Nuclear Medicine*

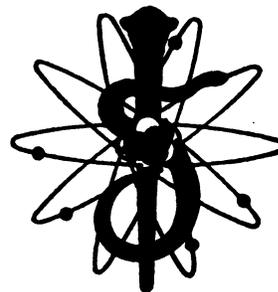

AN IMPORTANT NEW JOURNAL

featuring

Original articles in clinical medicine, basic and clinical medical research, physics and chemistry dealing with the use of isotopes in humans, and articles on related subjects. The latter includes dosimetry, instrumentation, protection, techniques, biologic effects contributing to the use or effects of isotopes in clinical medicine or the clinical effects of ionizing radiation.

George E. Thoma, M.D., St. Louis—Editor

G. O. Broun, Jr., M.D., St. Louis, Titus C. Evans, Ph.D., Iowa City,

Neil Wald, M.D., Pittsburgh, Eugene L. Saenger, M.D., Cincinnati—Associate Editors

The Journal of NUCLEAR MEDICINE

333 North Michigan Avenue, Chicago, Illinois 60601

\$20.00 per year, U.S. \$21.00 Foreign

Name.....

Address.....

City..... State..... Zip Code.....

Please remit by check or money order.

**Published
Monthly**



Announcement to Authors Preliminary Notes

Space will be reserved in each issue of THE JOURNAL OF NUCLEAR MEDICINE for the publication of one preliminary note concerning new original work that is an important contribution in Nuclear Medicine.

Selection of the preliminary note shall be on a competitive basis for each issue. One will be selected after careful screening and review by the Editors. Those not selected will be returned immediately to the authors without criticism. Authors may resubmit a rejected or revised preliminary note for consideration for publication in a later issue. The subject material of all rejected manuscripts will be considered confidential.

The text of the manuscript should not exceed 1200 words. Either two illustrations, two tables, or one illustration and one table will be permitted. An additional 400 words of text may be substituted if no tables or illustrations are required. Only the minimum number of references should be cited.

Manuscripts should be mailed to the Editor, Dr. George E. Thoma, St. Louis University Medical Center, 1402 South Grand Blvd., St. Louis, Missouri 63104. They must be received before the first day of the month preceding the publication month of the next issue, *e.g.*, preliminary notes to be considered for the November, 1965 issue must be in the hands of the Editor before October 1, 1965.

ADVERTISING INDEX

<p>Abbott Laboratories North Chicago, Illinois IFC, i</p> <p>Applied Health Physics Bethel Park, Pennsylvania vii</p> <p>Harshaw Chemical Co. Cleveland, Ohio xx</p> <p>New England Nuclear Corp. Boston, Massachusetts ix</p> <p>Nuclear-Chicago Des Plaines, Illinois BC</p> <p>Nuclear Consultants St. Louis, Missouri IBC</p> <p>Ohio-Nuclear Corp. Cleveland, Ohio xii</p>	<p>Packard Instruments Downers Grove, Illinois iii</p> <p>Phillips-Duphar Amsterdam, Holland viii</p> <p>Picker X-Ray Corp. White Plains, New York x</p> <p>Squibb, E. R. & Sons New York, New York xi</p> <p>Tracerlab Waltham, Massachusetts xvii, xx</p> <p>Volk Radiochemical Chicago, Illinois v</p>
--	---

HARSHAW NaI (Tl) CRYSTALS

pace Health Physics developments



IN WHOLE BODY COUNTING

- Crystals to 11½" Diameter
- Measured background spectrum
- Resolution and Stability Performance Guaranteed

IN SCANNING

- Best energy resolution ■ Multi-Crystal Mosaics for Dynamic Scanning
- Small Detectors
- Large Diameter Crystals for Pinhole Camera Techniques

IN GAMMA DOSE RATE DETECTORS

- New Development in Scintillator-Photoconductor Module for Dose Rate Measurement
- High Sensitivity ■ Fast Response Time

IN THERMOLUMINESCENT DOSIMETRY

- Activated LiF Offering ■ High Sensitivity
- Reproducibility ■ Linear Energy Response

IN CHARGED PARTICLE DETECTION

- Surface Barrier Diodes offering minimum size for Probe Devices
- Li Drift Detectors for Beta Detection

Detailed information can be obtained from:



Crystal-Solid State Division
THE HARSHAW CHEMICAL COMPANY
1048 East 97th Street • Cleveland, Ohio 44106 • Telephone 216 721-8300
Utrecht, Netherlands—Contact Harshaw-Van Der Hoorn N. V.
Frankfurt, W. Germany—Contact Harshaw Chemie GmbH



**the Tracerlab watchdog
stands guard over
more film badges than
any other monitoring service**

Every day of the week, every week of the year, the Tracerlab film badge service handles more dosimetry badges than any other service. Two reasons: complete customer confidence, computer-processing for sure-fire, super-fast returns. Turn to Tracerlab Twin-film badge service for exposure measurement of beta, gamma, neutron, x-ray or mixed radiation. They wrote the book.

And the Tracerlab watchdog sniffs out the data you may need in health physics: bioassay • environmental analysis • fission and corrosion products analysis • activation analysis • neutron absorption measurement • radioisotope applications • radiation warning labels, tapes and signs.

The Tracerlab watchdog stands for security. Count on him.



T R A C E R L A B

A Division of Laboratory For Electronics, Inc.
WALTHAM, MASSACHUSETTS 02154

- Film Badge Service • Health Physics • Bioassays • Sources
- Nuclear Instrumentation • Radiochemicals • Radioactive Waste Disposal
- Radiation Monitoring Instrumentation • Isotope Applications

For Greatly Improved Scanning

TECHNETIUM (Tc^{99m})

*Delivered
EVERY
Monday*

- an ideal scanning agent with easily collimated 140 kev gamma.
- 6 hour half-life minimizes patient exposure.
- permits dosage up to 10 mc, allowing greatly reduced scanning time with any scanner.
- can be administered orally or by intravenous injection.
- delivered EVERY Monday in NCC TechnoKow™ source generator.


*America's First Commercial
Supplier of Technetium (Tc^{99m})*

REFERENCES:

Harper, P.V., Beck, R., Charleston, D., and Lathrop, K.A.: "Optimization of a Scanning Method Using Tc^{99m}," Nucleonics, 22:1, 50-54, January 1964.

Harper, P.V., Lathrop, K.A., and Richards, P.: Tc^{99m} as a Radiocolloid. J. Nuclear Med. 5:5, 382, May 1964.

Smith, E.M.: Radiochemical Purity, Internal Dosimetry and Calibration of Tc^{99m} J. Nuclear Med. 5:5, 383, May 1964.

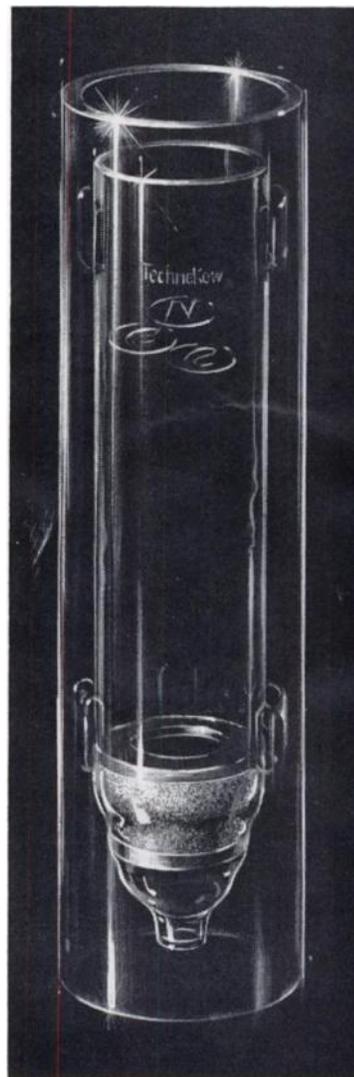


Write for complete information today



**NUCLEAR CONSULTANTS
CORPORATION**

Box 6172, Lambert Field, St. Louis, Missouri 63145

Chicago
Cleveland
Houston
Los Angeles
New York
San Francisco
Washington, D.C.

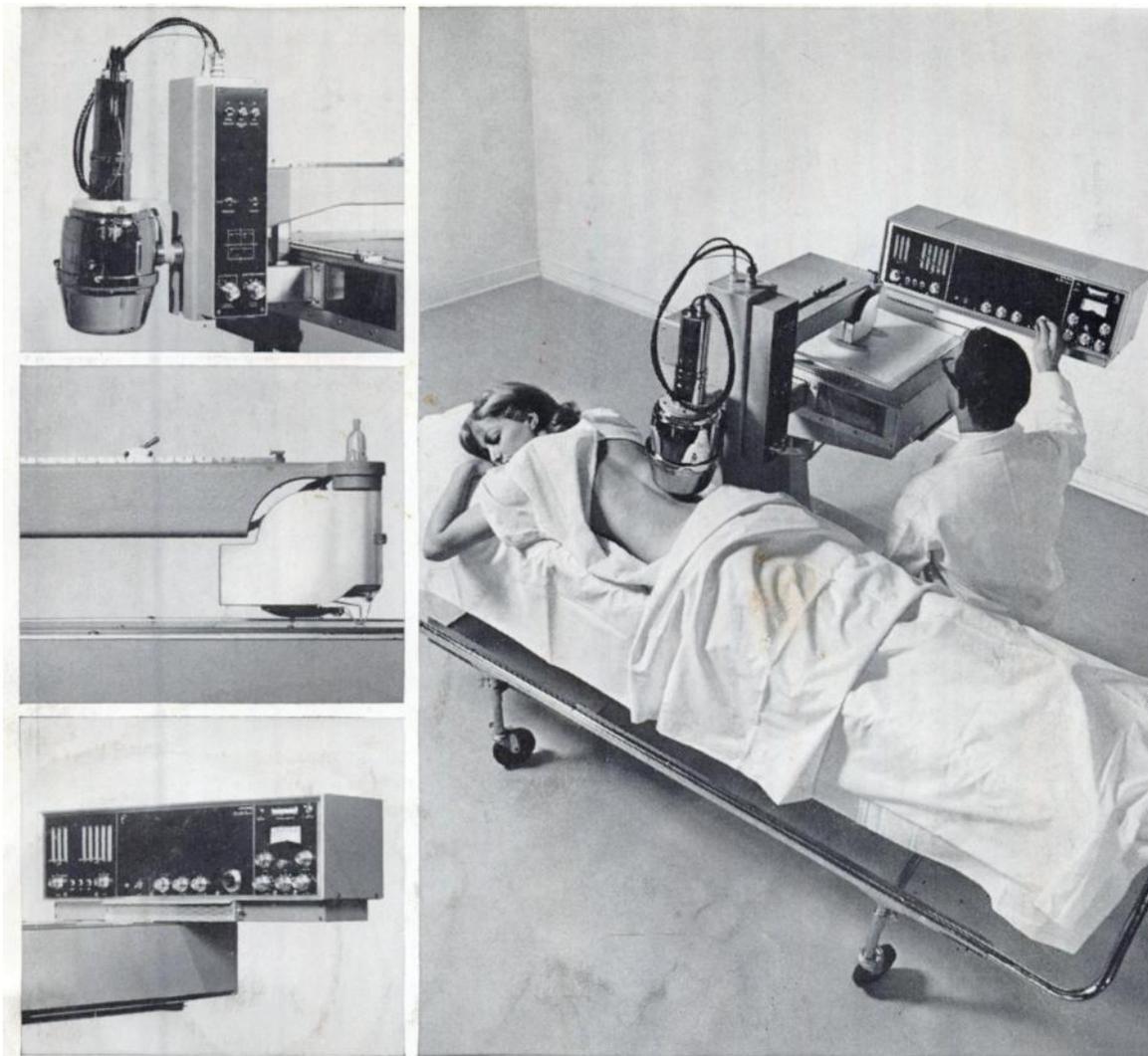

NUC-D-5-226

DOWNTIME: ZERO

CIRCUIT SLIP-UPS ARE OUT. The ultimate in reliability: operating circuits in Nuclear-Chicago's Pho/Dot® isotope scanner. Premium, ultra-stable, solid-state components. Heat-isolated critical circuits. Regulated power supplies.

RUGGED, TROUBLE-FREE CONSTRUCTION IS IN. In Pho/Dot, extra strength for rigidity and long-term operation. Scanning motions controlled by low-friction drives for reproducible results. Heavy-duty casters for mobility. Leveling jacks to simplify horizontal positioning.

SECOND-BEST IS OUT. Control of Pho/Dot quality every step of the way. Test after test of each subassembly. Exhaustive tests of the complete instrument. Precision and reliability built in—to stay.

DEPENDABLE SERVICE EVERYWHERE IS IN. The Pho/Dot guarantee: one-year, all-inclusive coverage of parts, labor, shipping, preventive maintenance, and travel time.

PHO/DOT FACTS. Many of your colleagues know. Your Nuclear-Chicago sales engineer can tell you even more about Pho/Dot's advanced design and foolproof operation. Or write us.



NUCLEAR-CHICAGO

A DIVISION OF NUCLEAR-CHICAGO CORPORATION
313 Howard, Des Plaines, Ill. 60018 U.S.A.

In Europe: Donker Curtiusstraat 7
Amsterdam W, The Netherlands

Scientists and engineers interested in challenging career opportunities are invited to contact our personnel director.